

# **PXR**

## ***The Digital X-Ray Pros***

### **Step for operation of X-Ray System**

#### ***Turning ON X-Ray system:***

1. Turn on **MAIN POWER** for x-ray System (located on the wall).
2. Turn on **X-RAY CONTROL POWER** switch (located on the X-Ray control, top/side).
3. Warm up X-Ray tube (refer to manufacture guidelines).
4. Adjust X-Ray techniques for patient study: begin study.

#### ***Turning OFF X-Ray System:***

1. Turn off X-RAY CONTROL POWER switch (located on the X-Ray control, top/side)
2. Turn off **MAIN POWER** for x-ray System (located on the wall).
3. X-Ray system is off.

### **Steps for operation of X-Ray Detector**

#### ***Turning ON Digital X-Ray Detector:***

1. Turn on **DIGITAL X-RAY POWER SWITCH** (Located on or near the front).
2. Turn on Computer and Monitor for digital x-ray.
3. Enter passwords and open software (Located on main desktop screen).
4. Input patient information, and capture image.

#### ***Turning OFF Digital X-Ray Detector:***

1. Turn off **PROCESSOR DETECTOR POWER SWITCH** (Located on or near the front).
2. Shutdown software and computer for monitor
3. X-Ray computer and detector is now off.

***PXR-The Digital X-Ray Pros***

***951.212.5110 (mon-fri 8am-6pm pst.)***